voco**lonolux®**

Application capsule

In accordance with DIN EN ISO 9917-2 : 2010

Instructions for use**Product description:**

lonolux is a light-curing, radiopaque glass ionomer restorative material in user-friendly capsules. The material has outstanding aesthetics, good handling properties and high chemical adhesion.

lonolux is available in shades A1, A2, A3, A3.5 and B1. The usable quantity of cement per capsule is at least 0.17 ml.

Indications:

- Class III and V restorations, primarily cervical fillings and root caries
- Restoration of deciduous teeth
- Small class I fillings
- Temporary fillings
- Core build-ups
- Linings

Application:**Preparation:**

In principle, cavity preparation should be minimally invasive to conserve healthy tooth substance. There is no need for undercuts.

Class V cavities should have a minimum depth of 1 mm. Remove preparation residues with water spray, then dry with air so that the surface remains slightly glossy (do not overdry). Place a matrix, if necessary.

If the residual dentine thickness is insufficient, the area close to the pulp should be covered with a calcium hydroxide product.

Activating the application capsule and mixing the material:

Only extract the application capsule from the tubular bag immediately before use. During activation, the liquid from the plunger must be pressed into the inside of the capsule as fully as possible. To achieve this, the end of application capsule where the plunger of liquid protrudes is slowly pressed down onto a hard surface (e.g. a table) as far as the plunger will move (see fig. 1). The capsule must only be activated directly before use and loaded immediately into a high-frequency mixer with approx. 4,000 oscillations per minute. Mix the activated application capsule for 10 s, remove it from the mixer without delay and insert it in the holder of an application device (e.g. AC Applicator, see corresponding instructions for use).

Application:

Carefully open the outlet by lifting the cannula until you feel a stop (see fig. 2), and apply the material directly into the cavity within 10 s of the end of mixing by compressing the lever on the applicator slowly and evenly. The cannula can be aligned as required by turning the whole application capsule inside the holder. Working time and consistency may be influenced by the respective type of capsule mixer and mixing time.

Fig. 1: Activating the capsule

Fig. 2: Opening the capsule

Avoid unnecessary waiting times between application and finishing since the setting process of the material may make it more difficult to work with.

Apply the selected shade of **lonolux** in layers with a maximum thickness of 2 mm, adapt with a suitable instrument and light-cure afterwards.

At room temperature, **lonolux** has a working time of 2.5 min. Higher temperatures shorten the working time, lower temperatures prolong it.

Light-curing:

Conventional polymerisation devices are suitable for light-curing the material. The curing time when using an LED-/halogen light with a minimum light-output of 500 mW/cm² is 20 s per layer.

Hold the light emission tip of the device as close as possible to the surface of the filling. If the distance is more than 5 mm, the curing depth may be reduced.

Quantity per capsule [ml]	min. 0.17
Net weight per capsule [g]	min. 0.35
Mixing time [s]	10
Working time [min]	min 2.5
Light-curing [s]	20

Finishing:

After light-curing, remove excess material and finish and polish the filling as usual.

Information, precautionary measures:

- Ensure that the application capsule is mixed immediately after activation.
- In case of contact with oral mucosa or skin, remove immediately with a cotton swab soaked in alcohol. Rinse with water.
- In case of contact with the eyes, rinse immediately with plenty of water and seek medical advice.
- **lonolux** contains polyacrylic acid, amines, BHT and methacrylates. **lonolux** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.
- Phenolic substances, especially preparations containing eugenol or thymol, disturb the polymerisation of the restorative cement. Do not use zinc-oxide eugenol cements or other eugenol-containing materials together with **lonolux**.

Storage instructions and application method:

Store **lonolux** in a dry place at temperatures of 4 °C - 23 °C. Only remove it from the foil immediately before use. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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