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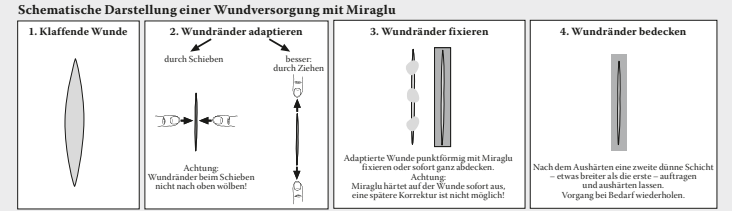
MIRAGLU®

Tissue adhesive PHIALS

GB Instructions for use – Please read carefully!

Composition: Every Miraglu phial contains 3g ethyl-2-cyanoacrylate tissue adhesive and is sufficient for the treatment of 20 wounds. Miraglu provides no culture medium for germs; it even has an inhibiting effect on existing germs.

Areas of application: Tissue adhesive for external use at the mouth and for application inside the oral cavity. For sealing wounds following carefully approximated wound edges, as a suture substitute, for additional stabilisation of sutures or as a protection against infection for wounds after sutures.

Contraindications: Neither must Miraglu be used on infected or inflamed wounds nor on bite wounds. Miraglu is non absorbable and must, therefore, not be used to close wounds of inner organs, the brain surface and central or peripheral nervous system. An application to these areas might damage the tissue and the subsequent scarring may cause complications. The application to the intima and media of blood vessels must also be avoided as vascular thrombosis and damage of the vascular wall may occur. Do not apply if the patient is allergic to cyanoacrylate or formaldehyde (see also: side effects). The adhesive may come off if the patient is heavily sweating. Miraglu must not be applied to the breast area during lactation, as the sucking motion could cause the adhesive to detach and enter the infant's mouth.

Precautions for the application and warnings: Bonds skin and eyelids within seconds. Therefore, avoid any contact with eyes and eyelids! Miraglu must not come into contact with the eyeball or conjunctival sac. Apply to wounds and the oral mucosa only. Ensure the exact adaption of wound edges prior to application. Keep surfaces to be adhered as dry as possible and fat-free. Liquid or pasty substances must not be applied before adhesion. Do not drip into the depth of wound, but apply superficially a maximum of 2 thin layers to skin surface. **Tip:** The fumes occurring during polymerisation of the adhesive might irritate the conjunctiva of the eye and the nasal mucosa. Therefore, we recommend protecting the patient's eyes and nose with moistened cotton wool.

First-aid actions to be taken in case of inadvertent contact of the adhesive with the eye: Serious mechanical damage to the cornea may occur if eye or lid-conjunctiva comes into contact with the adhesive. The affected eye must be rinsed with a sterile, saline solution (for approx. 15 min.) immediately after contamination. Should such a solution not be available, the eye may be rinsed with tap water. Application of sterile medical neutral oils may be advantageous.

General dosage instructions and method of application:

- Wait for complete haemostasis.
- Remove possible foreign objects from the wound.
- Clean the area around the wound with alcohol.
- Do not apply creams or unguents.
- Remove Miraglu from refrigeration or freezing.
- Always open only one capsule at a time, by pushing up the lid's recessed lip. By doing so, you tear off the tamper-proof safety-seal below kann, die im Extremfall zu Verbrennungen führen könnte.
- You can now take out the phial.
- If the material has been taken directly from refrigeration/freezing, please make sure to store the capsule in an upright position. Oth-herwise, condensation can collect in the capsule which could have a negative effect on the storage stability.
- Keep the phial upright and unscrew the cap of the phial's closure.
- Miraglu is now ready for use as the tip of the inner closure is open.
- Break off one drop-in-tray from the bar or open the sterile application aid.
- Dip the desired amount of Miraglu (approx. 1-3 drops) into the dent of the small drop-in-tray or the sterile pallet.
- Soak up the solution with the dispensing pipette by squeezing the pipette with thumb and index finger approximately 1 – 2 cm behind the bend.
- Dip the tip into the liquid and release your hold.
- Adapt the wound edges (see schematic diagram).
- Smooth out wrinkled skin before application.
- Blow the area that is to be treated dry as possible to avoid the adhesive from "floating" on the wound.
- Put some Vaseline onto your fingertip (above the glove). This makes sure the glove does not get stuck.
- Depending on the length of the wound, apply 1 – 2 drops (see schematic diagram).
- Spread out the adhesive over the complete length of the wound and also over part of the surrounding tissue, using the bend of the dispensing pipette. Make sure not to apply too much adhesive to the wound at once.
- Then immediately use the fingertip with the Vaseline to spread the adhesive again in order to achieve a smooth surface. This way, there will not be any hard adhesive sticking out.
- Wait until the first layer of adhesive has hardened. Takes about 10 seconds.
- You recognise the polymerisation by the lessened glossy appearance of the adhesive
- Apply again 1 – 2 drops of adhesive and spread the liquid in a wider layer than the first time – the adhesive must be spread thinly onto the healthy skin.
- Wait again until the adhesive has set.
- The final polymerisation of several adhesive layers can take a few minutes.
- A later correction of the wound edges is only possible by stripping off the totally polymerised adhesive.
- Dispose of drop-in-tray, pallet and dispensing pipette after the tissue adhesive has polymerised.
- Screw on the upper cap of the phial closure, put the phial into the capsule, close it with the lid and store the capsule in an upright position.
- Always store the capsules with the phials (unopened or opened) in the refrigerator (see: storage instructions).
- Before storing, check if condensation has formed in the capsule (by direct removal from the cooling). In this case, it is essential to dry the capsule as the liquid can have a negative effect on the storage stability.

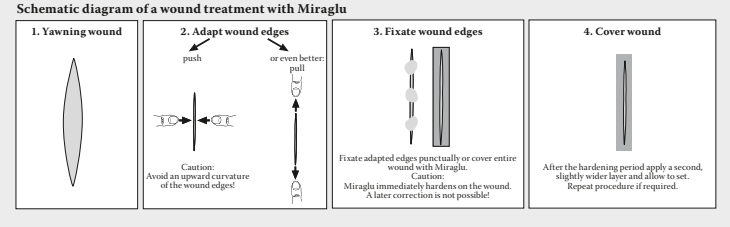
Special notes:

1. Please note that all acrylate adhesives, including Miraglu, due to the underlying wet environment do not adhere as long in the mouth as on treated areas of the skin.
2. Since, however, the wounds are deprived of disturbances by movements of the tongue, saliva, or food, they heal considerably faster as untreated wounds or treatments made with other materials.
3. At the following check-up it can be decided whether Miraglu should be re-applied.
4. Miraglu is non-absorbable and must only be applied to the surface.

Important: Apply the adhesive sparingly and smooth it out to a very thin layer that covers the complete wound and also over part of the healthy tissue. It is not advisable to apply one thick layer, as this does not improve the adhesive strength, but causes the adhesive to peel off from the healthy skin more easily. Avoid contact of the adhesive with gloves, instruments, gauze, etc. to prevent adhesion.

Important information and tips for the use of the capsules and phials:

Open only one capsule at a time. Do not, under any circumstances, remove the hygroscopic material in the lid of the capsule as it ensures for a dry environment inside the capsule. Do not store phials without the capsule and always close the capsule with the lid. Immediately after use, close the phial. Screw the top cap back onto the phial, making sure the thread is not contaminated with the adhesive. Otherwise, it could be impossible to open the closure for future use.

The wound is now closed and secure from infection. The patient can now return to his usual hygiene habits.

Caution: The Miraglu layer must never be covered with a dressing of any kind, as long as the adhesive is not completely set. Otherwise an increased thermal reaction (heat development) can occur, which in extreme cases could lead to burning.

If a "throbbing" sensation occurs a few hours after the wound has been adhered, it is advisable to peel off the adhesive and to treat the wound in a different way. It can be assumed that at the time of adhesion the wound was already inflamed or that foreign bodies were present. If it is determined during a check-up that not enough Miraglu has been applied, and the material detaches itself from the surrounding healthy skin, another layer of Miraglu can easily be re-applied, as Miraglu provides no culture medium for germs and even an inhibiting effect on existing germs. After 5 – 10 days, when the healing process is advanced, Miraglu detaches itself like scab. In most cases further treatment of the wound is unnecessary.

Side effects: If the adhesive is applied improperly or too thickly, thermal tissue damage may occur due to the polymerisation process. An extensive, too concentrated layer prevents the healing of connective tissue. No allergic reactions have been reported up until now. Local tolerance is excellent if the product is used according to the instructions. Do not use if the patient is hypersensitive to cyanoacrylate or formaldehyde (also see: Contraindications). Should Miraglu unintentionally come into contact with healthy skin and polymerise, it will detach itself after a while without causing damage to the skin.

Storage instructions and shelf life of Miraglu: Follow the storage instructions on the package and capsules. Store Miraglu in the deep freezer (-18°C). Miraglu is immediately ready for use when taken out of the deep freezer as it does not freeze. The expiry date is stated on the capsule as well as on the outer package. Do not use Miraglu after the stated expiry date. Polymerisation of the material has begun if the solution does not drip from the phials any longer but shows a stringy consistency instead. In that case, an exact adaption is no longer possible and the adhesive must no longer be used. Store opened phials inside the capsules with the closure in an upright position. The mentioned maximum shelf life can only be guaranteed in the deep freezer (-18°C).

Properties: The monomeric, liquid form of ethyl-2-cyanoacrylate rapidly turns into the polymeric form when in contact with water or tissue fluids. Even minimal quantities of Miraglu bond the treated surfaces together.

Notes for medical staff: Hazard warnings: Causes skin irritation. Causes serious eye irritation. May cause respiratory irritation.

Safety notes: Avoid breathing fumes. Wear protective gloves/protective clothing/eye protection/face protection. Wash thoroughly after handling. Must only be used outdoors or in well-ventilated rooms.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. Take off contaminated clothing and wash before reuse.

IF INHALED ACCIDENTALLY: Remove victim to fresh air and keep at rest in a position comfortable for breathing. Store locked up. Store in a well-ventilated place. Keep container tightly closed.

Disposal: Disposal of content/of the container in line with local/regional/national/international regulations.

Cyanoacrylate: Danger. Bonds skin and eyelids within seconds. Keep out of reach of children.

Re-order items/accessories:
Miraglu set: 1 tube with 3 g, 20 fine dosing pipettes 2 pallets
Miraglu fine dosing pipettes 100 pieces
Miraglu mixing pallets single

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HAGER & WERKEN GmbH & Co. KG