DENTAL LIGHT-CURED REPAIR KIT

CLEARFIL™ REPAIR

INSTRUCTIONS FOR USE

MODE D'EMPLOI

MODE DE EMPLEO

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KURARAY MEDICAL INC.

CONTENTS

ENGLISH 1～6

FRANÇAIS 7～12

ESPAÑOL 13～18

ITALIANO 19～24

NEDERLANDS 25～30

DEUTSCH 31～36

ENGLISH

I. Description

CLEARFIL™ REPAIR is a light-curing bonding system for porcelain, all ceramics or composite resin repair of their fractured facing crowns and bridges. Besides, this system is used for aesthetic repair intraorally.

II. Components

CLEARFIL™ REPAIR consists of the following components:

CLEARFIL™ SE BOND PRIMER

This primer is a self-etching primer and offers simultaneous treatment of both enamel and dentin. This primer contains a phosphate monomer (MDP). As a result, it produces high bond strength to not only tooth structure but also the metals. Besides, the mixture of PRIMER and CLEARFIL™ PORCELAIN BOND ACTIVATOR can be used as a treatment liquid of porcelain, composite or all ceramics surface.

CLEARFIL™ SE BOND BOND

This BOND contains a phosphate monomer (MDP), methacrylate monomer (HEMA) and small amount of inorganic filler. Therefore, this bonding agent shows high bond strength to tooth structure and metals. Moreover, after polymerization, the bonding layer offers superior mechanical properties compared with the bonding agent without inorganic filler.

CLEARFIL™ PORCELAIN BOND ACTIVATOR

CLEARFIL™ PORCELAIN BOND ACTIVATOR is silane coupling agent using conjunction with CLEARFIL™ SE BOND, CLEARFIL™ LINER BOND 2V, CLEARFIL™ PHOTO BOND or CLEARFIL™ NEW BOND.

ALLOY PRIMER

This metal adhesive primer contains a phosphate monomer (MDP) and precious metal adhesive monomer (VBATDT). Therefore, this metal adhesive primer provides high bond strength to both precious and base metals.

K-ETCHANT GEL

This material is a phosphoric acid etching liquid for an adherent surface.

CLEARFIL™ ST OPAQUER

CLEARFIL™ ST OPAQUER is low viscosity composite resin to mask an exposed metal surface. And this material shows high opacity. Therefore it is necessary to apply a visible light for 40 seconds.

III. Indications

CLEARFIL™ REPAIR is indicated for the following cases:

- Intraoral repairs of fractured porcelain or composite facing crowns/bridges.
- Intraoral repairs of fractured all ceramics restorations.
- Intraoral repairs of fractured porcelain and composite inlays/onlays.

IV. Contraindications

- Patients with a history of hypersensitivity to methacrylate monomers.

V. Possible side effects

The mucous membrane may turn whitish when contacted with CLEARFIL™ SE BOND PRIMER due to the coagulation of protein. This is a temporary phenomenon that will disappear in a few days.

VI. Precautions

Safety precautions

1. Avoid use of the product for patients with a history of hypersensitivity to methacrylate monomers.
2. If any hypersensitivity such as dermatitis occurs, discontinue use of the product and consult a physician.
3. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect the patient's eyes from splashing material.
4. If the product comes in contact with the human body, take the following measures:
 - <If the product gets in the eye>
Immediately wash the eye with copious amounts of water and consult a physician.
 - <If the product comes in contact with the skin>
Immediately wipe it off with a cotton pledget moistened with alcohol or gauze and wash with copious amounts of water.
5. Use caution to prevent the patient from accidentally swallowing the product.
6. Avoid looking directly at the curing light when curing the product.
7. Keep the product out of reach of children.

Handling and manipulation precautions

1. CLEARFIL™ SE BOND PRIMER and CLEARFIL™ SE BOND BOND polymerize by visible light (especially ultraviolet rays). Use the light blocking plate to avoid operating light or natural light (sunlight from windows) and use within 3 minutes from dispensing.
2. The emitting tip of the visible light-curing activator should be held as near and vertical to the resin surface as possible. If a large resin surface is to be light cured, it is advisable to divide the area into several sections and light-cure each section separately.
3. Low intensity of light causes poor adhesion. Check the lamp for service life and the visible light guide tip for contamination. It is advisable to check the light intensity of light-curing activator using an appropriate light-checker at periodic intervals.

Storage precautions

1. Do not use after the expiration date. Note expiration date on the outside of package.
2. The product should be refrigerated when not in use; bring it to room temperature before using.
3. Keep away from extreme heat or direct sunlight.
4. Replace the bottle cap as soon as possible after resin has been dispensed from the bottle.

VII. Clinical procedures

1. Preparation of fractured surfaces


- 1) Facing material surface
Using a diamond point, remove a layer of the fractured surface, and place a bevel at the marginal area.
- 2) Metal surface
Roughen the metal surface with a diamond point.

2. Acid etching of facing material surface

1. Apply K-ETCHANT GEL to the facing material surface and leave it in place for 5 seconds before washing and drying. If the adherent surface extends to uncut enamel, apply K-ETCHANT GEL and leave it in place for 10 seconds before washing and drying.

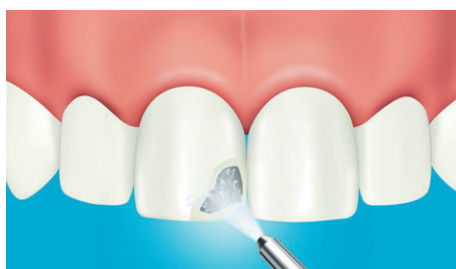2. After that, thoroughly rinse with water.

3. Dry with oil-free air.

4. Precious metal surface

When precious metal is used, apply ALLOY PRIMER according to the manufacturer's instructions for use.

3. Silane treatment

1. Dispense one drop each of CLEARFIL™ SE BOND PRIMER and CLEARFIL™ PORCELAIN BOND ACTIVATOR into a well of the mixing dish and mix them.

[NOTE]

CLEARFIL™ SE BOND PRIMER and CLEARFIL™ PORCELAIN BOND ACTIVATOR should be mixed immediately before application.

2. Apply mixture to the facing material and metal surfaces with a sponge or a disposable brush tip. Leave it in place for 5 seconds. Use caution not to allow saliva or exudate to contact the treated surfaces. If the adherent surface extends to the tooth, leave mixture in place for 20 seconds.

3. After applying CLEARFIL™ SE BOND PRIMER, evaporate the volatile ingredients with a mild oil-free air stream. The treated metal surface must be sufficiently dried; if not, adhesion might be seriously impaired.

[CAUTION]

Do not wash treated surface. If the treated surface is contaminated with saliva, wash it with water, dry, or clean with alcohol, and treat with the mixture again.

4. Bonding


1. Dispense the necessary amount of CLEARFIL™ SE BOND BOND into a well of the mixing dish.

[CAUTION]

BOND polymerizes by visible light (especially ultraviolet rays). CLEARFIL™ SE BOND BOND will set to a gel if left under the operating light or natural light. Use the light blocking plate to avoid operating light or natural light (sunlight from windows) and use within 3 minutes from dispensing.

2. Apply CLEARFIL™ SE BOND BOND to the entire cavity wall with a sponge or a disposable brush tip.
3. Use a light air stream to make the bond film as uniform as possible.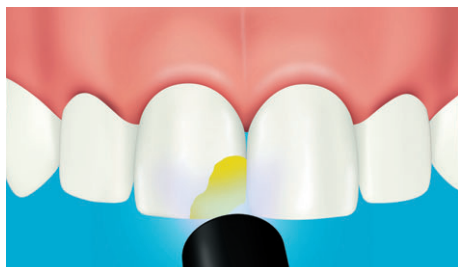


4. Light-cure CLEARFIL™ SE BOND BOND for 10 seconds with a visible light-curing activator.

5. Applying the opaquer

1. Use opaque resin such as CLEARFIL™ ST OPAQUER for metal surface to prevent metal shine through.

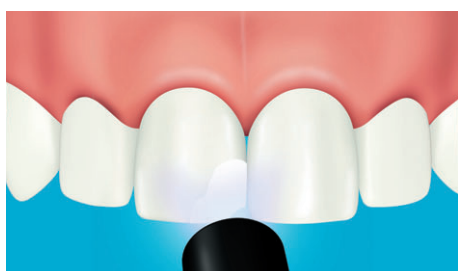2. Light-cure the composite for 40 seconds.

6. Light-curing composite filling

1. Place composite (e.g. CLEARFIL™ AP-X) into the cavity, light-cure, finish and polish according to the manufacturer's instructions for use.

[WARRANTY]

KURARAY MEDICAL INC. will replace any product that is proved to be defective. KURARAY MEDICAL INC. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

[NOTE]

CLEARFIL are trademarks of KURARAY CO., LTD.

Manufactured by
KURARAY MEDICAL INC.
1621 Sakazu, Kurashiki, Okayama 710-0801, Japan
European Union Representative
KURARAY EUROPE GmbH
Building F821, Hoechst Industrial Park
65926 Frankfurt am Main, Germany
Tel:+49 (0)69 305 35 840 Fax:+49 (0)69 305 35 640

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CLEARFIL sind eingetragene Warenzeichen von KURARAY CO., LTD.