## Albédo

v. N50.0 ON/OFF

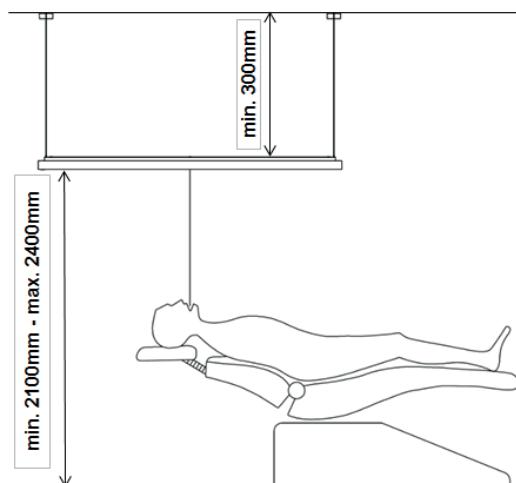
v. N50.R Regul. Connect.

v. D65.0 ON/OFF

v. D65.R Regul. Connect.


### FR INSTALLATION

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
### EN INSTALLATION
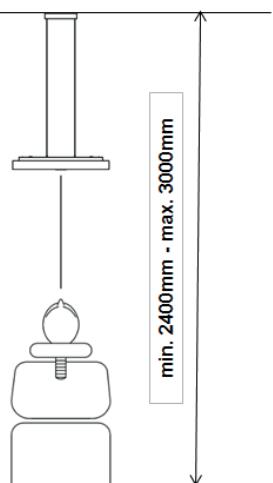
#### LIGHTING FIXTURE POSITIONING

### DE MONTAGE

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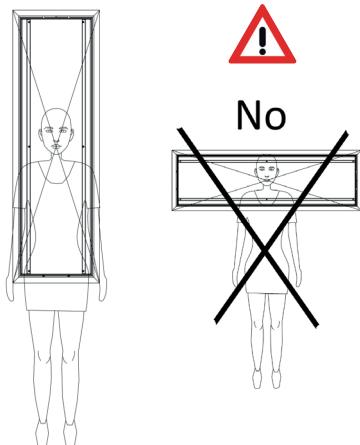### PREAMBULE

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### PREAMBLE

The installation instruction does not supersede local and (inter)national regulations for electrical installations.

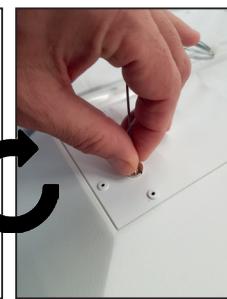
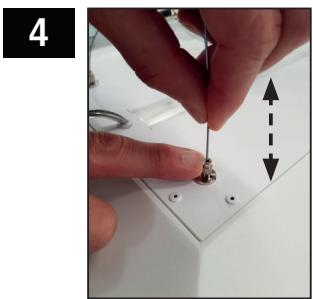
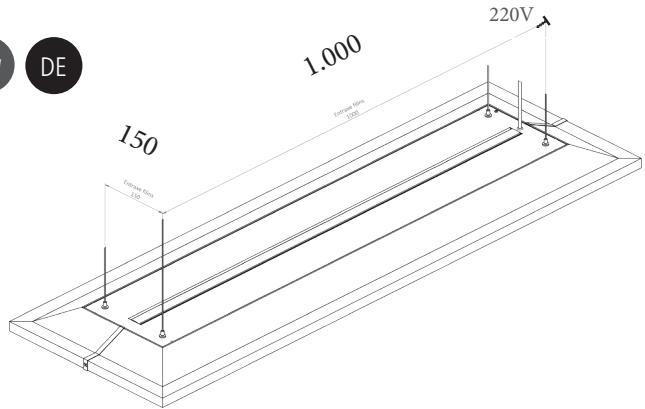
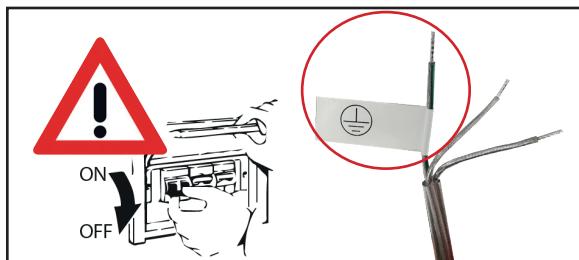
- The product must be installed following applicable and appropriate electrical standards and according to the instructions provided by degré K in order to avoid any risks of fire or electrical shock.
- Before installation, maintenance or cleaning, always first turn off power to the circuit.
- Strictly respect the Earth wiring** (wire with discreet green-yellow border, marked with a label).
- Carefully unpack the product.
- Only the exchange of a Driver for an original part is authorized. **Never attempt to access the interior of the luminaire since doing so may void the warranty.**
- Contact Degré K or the local reseller for technical assistance.
- Keep at an ambient temperature of: -20°C to +45°C.

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## EN UNDER DROPPED CEILING

If mounting on a false ceiling with slabs, see procedure for inserting the cables.

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## EN CLEANING

- The light's diffusers are resistant to non-aggressive disinfectant solutions.

Clean with a soft disinfection wipe type Durr FD366, rubbing only on the LED-to-LED AXLE.

After the disinfection time has passed, wipe down the light using a clean, SOFT MICROFIBER cloth respecting the direction of the LEDs.

- The frames are sufficiently resistant to alcoholic solutions type FD333 and FD322 but the paint finish of the frames may be damaged by the use of other types of liquid cleaning solutions.
- The paint finish of the frames is sensitive to extreme scrubbing.

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