Scan for more information

Check out our learning platform Reliance U!
www.RelianceU.tech

Reliance Orthodontic Products, Inc.
1540 W. Thorndale Avenue,
Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com

EC REP MD

EMERGO EUROPE

Prinsesgracht 20, 2514 AP The Hague,
The Netherlands

AUSTRALIAN SPONSOR EMERGO AUSTRALIA

Level 20, Tower 1, Darling Park, 201 Sussex St,
Sydney, NSW 2000, Australia

ETCHMASTER®

INSTRUCTIONS FOR USE

V2 MAR.2022

RELIANCE®
ORTHODONTIC
PRODUCTS

RelianceOrthodontics.com

EN-ENGLISH

GENERAL INFORMATION - The EtchMaster® SmartTip® is a disposable single-use self-contained microetching device that is powered by house air available at every dental chair. Mount the adapter to chairside as required by the adapter connection type. Refer to User Instructions for SmartTip attachment and removal directions.

Please Click here: www.relianceorthodontics.com web site for SDS.

The SmartTip may be utilized as a continuous sweep over the target area or it is a pulse sweep. Higher operating pressure is recommended for difficult sweeping sites. For safety, efficiency and cost considerations operating 40-45 psi is recommended and should NOT be exceeded. During operation noise is held about 1mm-2mm from the target surface holding the instrument further away from the surface increases the size of the treated area.

INDICATIONS FOR USE - With Hard Abrasives

Microetching of tooth surfaces (enamel & dentin), general purpose bonding and orthodontic bonding procedures, including orthodontic brackets, ceramic and porcelain repair, crown & composite restorations, CEREC®, Zirconia, elox, Lava, Veneers, Matrix bands, and Powders. Polishing, cleaning, polishing, desensitizing, prophylaxis, preventive sealants, superficial & ingrained stain removal, pits & fissure brushing, subgingival biofilm removal from implant surfaces.

SAFETY: Read the instruction carefully before using this device. The power source must be connected to the Reliance Orthodontic Products, Inc. web site. The manufacturer, distributor, or retailer of this product can exercise no control over the use of the EtchMaster®.

- This is a disposable device. Dispose after single patient use regardless of the remaining quantity of powder.

- DO NOT attempt to sterilize, re-use or dispose the tip.

- Mounting and pressurizing the device more than once will degrade device operation.

- Federal Law restricts this device to sale by or on the order of a licensed dentist.

- Rubber dam must be used to protect adjacent teeth and soft tissue. Matrix band should be used to protect adjacent teeth and soft tissue.

- Due to the aluminum powder, it is necessary to protect the eyes and fingers present in the treatment room during the procedure, namely the patient, assistant and dentist. For patient protection goggles and/or face (eyes and nose) covering with powder resistant material is recommended. Dental personnel should already wear protective eyewear. We recommend dental personnel wear face shields over safety glasses and masks for extra protection.

- Do not use oxygen or other flammable or toxic gases as propellant.

- Excessive microetching will erode most surfaces. Practice and experiment on typodonts, pennies, stainless steel, extracted teeth, or glass slides before actual use. These surfaces will simulate both precious and non-precious alloys and porcelain.

CAUTION: DO NOT POINT THE ETCHMASTER® SMARTIP® NOZZLE TOWARD YOUR FACE OR EYES. ALWAYS WEAR PROTECTIVE SAFETY GLASSES WHEN USING THIS DEVICE. NEVER REFILL AND/OR REUSE THE DEVICE ONCE IT HAS RUN OUT OF MATERIAL. SAFELY DISPOSE AFTER USE.

DISINFECTION: Groman Dental recommends using a disinfectant on the adapter body between patients or using a single use disposable barrier. To disinfect the EtchMaster® adapter generously spray disinfectant on a clean lab and wipe all surfaces. Discard used towel. Dry with a clean cloth. Never spray disinfectant solution directly on the adapter. When using a plastic sleeve over the EtchMaster® adapter, spray disinfectant solution directly on the plastic sleeve and not contact to the adapter. Although the EtchMaster® adapters are sterilizable to 135C degrees autoclave, this is not necessary for this item. Do not immerse handpieces in liquid.

USER INSTRUCTIONS

IMPORTANT - REVIEW SAFETY PRECAUTIONS PRIOR TO USE. Manufacturer is not responsible for damages due to misuse of the product.

STEP 1 - Check SmartTip color code mark to identify powder type

BLACK- 50 micron aluminum oxide Hard Abrasive

BROWN- .90 micron aluminum oxide Hard Abrasive

Note: 90micron powder is coarse and not recommended for intrasurgical use.

(Check manufacturer's color code chart for other powder types)

STEP 2 - Hold tube end of the SmartTip with one hand and the handheld adapter with the other and firmly insert tube end into the adapter collet.

IMPORTANT - After insertion pull on SmartTip to ensure secure engagement with the adapter collet.

STEP 3 - Twist and Pull SmartTip nozzle forward until stop to unseat the abrasive powder contained within the SmartTip.

SmartTip is now operational.

STEP 4 - Direct SmartTip nozzle at target surface and apply air flow.

Discard EtchMaster® Post-Op or once powder is depleted.

STEP 5a - Point adapter downward to remove SmartTip.

STEP 5b - Remove SmartTip from adapter by retracting collet while twisting and pulling SmartTip away from the collet.

NOTE: Report any sensitivity to Reliance and the competent authority of the state in which the end-user patient is established.

To READER: #ENTL, #ENTL, #ENTL, #EMTLL, #EMTRL, #EMTRSS, #EMTRSO, #EMTRAS, #EMMAS, #EMTR90

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