COLTENE

BioSonic® UC31

General Purpose Cleaner + Super Rust Inhibitor

DILUTE 1:2 - 4 L (0.5-1 gal) **CONT** 20 x 15 ml packets **REF** UC31

Rx ONLY

Made in USA

SDS www.coltene.com

Coltène/Whaledent Inc. 

235 Ascot Parkway
Cuyahoga Falls, OH 44223 / USA
Tel. USA & Canada

+1 800 221 3046

+1 330 916 8800

Fax +1 330 645 8704

info.us@coltene.com

Coltène/Whaledent GmbH + Co. KG **EC REP**

Raiffeisenstrasse 30
89129 Langenau / Germany
Tel. +49 (0)7345 805 0
Fax +49 (0)7345 805 201
info.de@coltene.com

P/N 20005564A

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

Intended Use – The BioSonic® Super Concentrated General Purpose Cleaning solutions is a concentrated nonionic, non-enzymatic, chemically neutral ultrasonic cleaning solution with corrosion inhibitor for dental and orthodontal instruments.

Intended Users – the UC31 is intended to be used by dental professionals.

Intended Patient Population – the UC31 is a non-patient contacting medical device.

Warnings and Precautions –

	DANGER: Contains 7.5% Disodium Tetraborate Pentahydrate: May damage fertility or the unborn child.
	DANGER: Contains 7.5% Disodium Tetraborate Pentahydrate: Causes serious eye irritation.

Note: when diluted according to the directions for use, this product is not considered hazardous.

Contraindications – There are no known contraindications.

Residual Risks, Side Effects –

- Failure to follow instructions for use may result in user harm and may decrease the effectiveness of the device.
- Failure to follow the instructions provided for changing solution may lead to ineffective cleaning or cross contamination that may have adverse effects on the patient.
- The solutions are intended to be diluted. Contact with concentrated solution may result in eyes, skin, lung, or digestive tract irritation.
- Using this solution past the listed expiration date may result ineffective cleaning that may have adverse effects on the patient.

Clinical Benefits/Performance Characteristics – The Biosonic® UC31 General Purpose Cleaner and Super Rust Inhibitor is a concentrated nonionic, non-enzymatic, chemically neutral ultrasonic cleaning solution that aids in the cleaning of dental and orthodontal instruments. By increasing cavitation in the BioSonic® Ultrasonic Cleaners, the UC31 aids in removing particulates. Each unit dose packet makes 2 to 4 liters of solution.

Prevention: Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Use personal protective equipment as required, safety glasses/goggles and protective gloves. Wash hands thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection. If exposed or concerned, get medical advice/attention. If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing for at least 15 minutes. If eye irritation persists, get medical advice/attention.

Storage and Disposal – Store Locked up. Keep out of reach of children. Store above 4 degrees Celsius (40 degrees Fahrenheit). Dispose of in accordance with federal, state, and local regulations.

DIRECTIONS FOR USE:

- Cut or tear packet at notched tip end
- Dispense liquid into 2-4 liters of water

Approx. Tank Size	Fill Capacity	Concentrate Needed
0.8 Gallon (3.02 L) BioSonic® UC50	0.5 Gal. (2.0 L)	1 packet
1.25 Gallon (4.73 L) BioSonic® UC125	1 Gal. (3.8 L)	1-2 packets
1.75 Gallon (6.6 L) BioSonic® UC150	1.5 Gal. (5.7 L)	2-3 packets
3.5 Gallon (13.25 L) BioSonic® UC300	3 Gal. (11.4 L)	3-6 packets

It is recommended to change the solution every 8 hours at a minimum or whenever the solution becomes visibly soiled or dirty.

Notice: Report any serious incident that has occurred in relation to this device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Symbols Glossary

Symbol	Meaning
	Consult the instructions for use
	Medical Device Indicator: this product meets the definition of a medical device
	Sale Restricted: the purchase of the device is restricted to healthcare practitioners only
	Use by date
	COLTENE catalog number
	CE Mark
	Indicates the entity importing the medical device
	The entity that manufactures the medical device
	Batch code
	Temperature - Low Limit Indicates the lowest temperature to which the medical device can be safely exposed
	Authorized Representative in the European Community
	Consult safety data sheet

DILUTE	Dilution ratio
CONT	Contents
	DANGER: Contains 7.5% Disodium Tetraborate Pentahydrate: Causes serious eye irritation.
	DANGER: Contains 7.5% Disodium Tetraborate Pentahydrate: May damage fertility or the unborn child.

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