GLASIONOMER CEMENT CX-PLUS

ENGLISH INSTRUCTIONS FOR USE

Glaslonomer Cement CX-Plus is a radiopaque glass ionomer cement recommended for cementing crowns, bridges, inlays or orthodontic appliances securely in place. Special features include: Excellent handling properties, low film thickness, less moisture sensitivity during the initial setting stage and prolonged working time followed by snap-set reaction. It releases fluoride.

1. Powder/liquid proportion : One level scoop of powder with two drops of liquid gives a powder to liquid ratio of 2.0g powder to 1.0 g liquid.
2. Dispensing: Use the green Shofu powder scoop and dispense one level scoop of powder (using the ledge inside the powder bottle to level off) onto a glass slab or mixing pad. Remove cap from liquid bottle and dispense two drops of bubble-free liquid (remove any air-bubbles trapped in the neck of the bottle by holding it vertically and gently squeezing the bottle, beforehand) near the powder, then replace the caps back onto the bottles.
3. Preparation: Thoroughly clean the preparation with oil-free pumice paste. Rinse with water and air dry. If there is near, or total, exposure of the pulp, line with Calcium Hydroxide before cementation. Clean the surfaces (of the post, crown, bridge etc.), that are to be cemented to the tooth structure, in an ultrasonic bath, rinse with water, wipe with pledget moistened in alcohol, and then air dry.
4. Mixing / Mixing times: Mix on a mixing pad or glass slab at room temperature (70-77° F / 21-25°C), in a humidity range of 50+/-10%. Mix together the dispensed powder and liquid until homogeneous, by applying short, rapid strokes with a plastic spatula. (Because of the excellent integration of the powder and liquid, it is not always necessary to divide the powder up into portions for gradual mixing). Total mixing time should be 30 seconds.
5. Placement: Use the spatula to apply a thin lining of cement to the internal surface of the crown. Do not overfill with cement. Seat the interfacing surfaces and have the patient bite down reasonably hard on a cotton ball to remove air and excess mixed cement. Care should be taken to protect the cement from moisture (water or saliva) particularly at the margins.
6. Working and Setting times: Working time is about 3.5 minutes from start of mix. The net setting time is 4 minutes from end of mix. For a longer working time, pre-chill the powder and liquid and use chilled glass slab to mix on.
7. Storage: Since this cement is slightly hygroscopic, reseal the powder and liquid bottles immediately after use. Store both in a cool place and do not subject to excessive sunlight or heat.

Note: Avoid contact of powder and liquid with the eyes. In case of contact, wash thoroughly with water and obtain medical advice.

Note:

- *Do not use after expiry date.
- *This product is only to be used for recommended indications.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional

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Manufactured by
SHOFU INC. 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan

SHOFU INC. Singapore Branch 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684

SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, California 92078-4059, USA

SHOFU UK Riverside House, River Lawn Road, Tonbridge, Kent, TN9 1EP, UK

SHOFU DENTAL TRADING (SHANGHAI) CO., LTD. No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China

EC REP SHOFU DENTAL GmbH Am Brüll 17, 40878 Ratingen, Germany

www.shofu.de