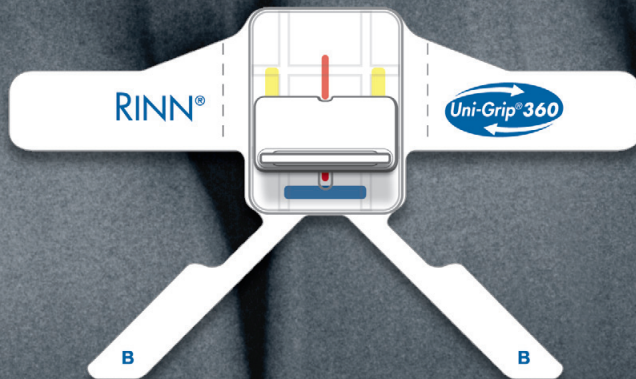
UNI-GRIP® 360

UNIVERSAL SENSOR HOLDER

- 550052 Uni-Grip® 360 50-Pack
50 Uni-Grip 360 Universal Sensor Holders
and 50 Sensor Covers
- 550252 Uni-Grip® 360 Combo Pack
100 Uni-Grip 360 Universal Sensor Holders,
100 Sensor Covers, and 1 Uni-GripAR
Positioning Arm and Ring
- 550100 Uni-GripAR® Positioning Arm and Ring
- 550500 Universal Sensor Covers, package of 500

Made in USA

For better dentistry

DENTSPLY
RINN

Caution

U.S. Federal laws restrict this device to sale to or on the order of a dentist.

Indications for Use

RINN Uni-Grip® 360 Universal Sensor Holder is indicated for use as a digital sensor holder during intraoral radiographs.

Contraindications

None known.

Warnings

Uni-Grip® 360 is single use only. Do not reuse.

Precautions

To be used only by individuals professionally trained to take intraoral radiographs. During use wear protective gloves. When using with digital sensor, only apply to protective sensor cover. Do not pull on sensor cord when removing. Only apply to dry surfaces.

Adverse Reactions

None known.

Storage

Store at room temperature (65°F to 75°F), out of direct sunlight.

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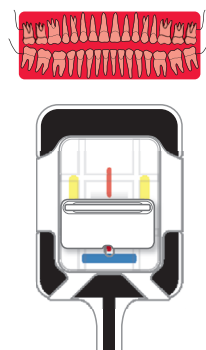
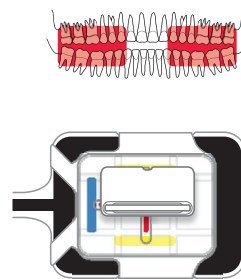
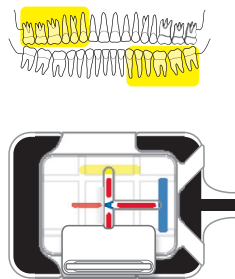
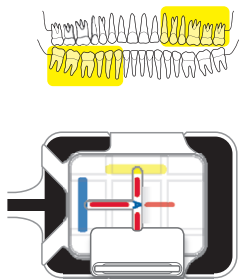
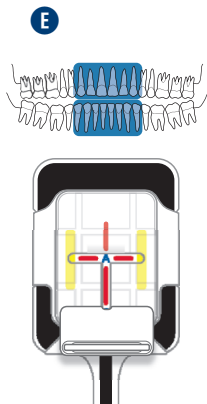
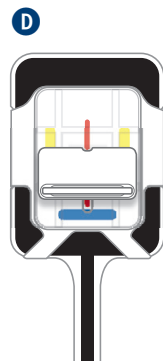
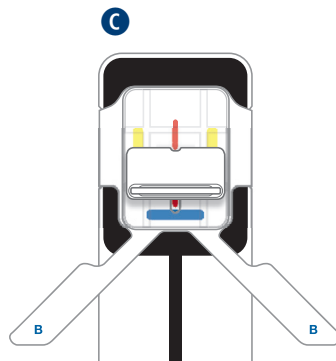
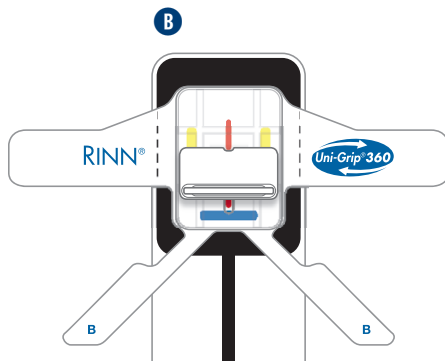
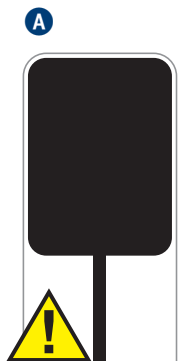
Step-By-Step Instructions

1. Cover digital sensor with sensor cover included in the Uni-Grip® 360 package. **A**
2. Remove backing liner from Uni-Grip® 360 and apply to center of active area with adhesive tabs marked **B** towards the bottom of the sensor where cord exits **B**.
3. Tightly wrap side tabs first **C** and then bottom tabs to back of sensor **D**.
4. Slide bite tab to desired position **E** and take x-ray. Repeat as necessary.
5. To remove, lift base to separate perforations on side tabs **F** and then remove bottom tabs. Do not pull on sensor cord when removing sensor from sensor cover.

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Usage Tips

1. The Uni-Grip® 360 bite-tab can rotate anywhere on the base. If the bite-tab becomes separated from the base, it can be reinserted at point **A**. **G**
2. The bite-tab can extend past the edge of the base for a wide range of positioning options. To capture more area past the apices, make sure tab is fully extended past base **H**. To capture more of the incisal edge, the bite-tab can be adjusted up on base **I**.
3. For maximum patient comfort, always place sensor in the center of mouth.
4. Instruct the patient to "bite lightly" to hold sensor in place.
5. For additional assistance in the United States, call RINN Customer Service at 800-323-0970. A training video is also available at www.UniGrip.com.

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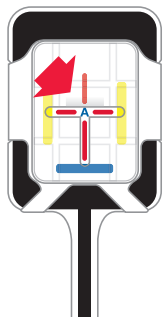
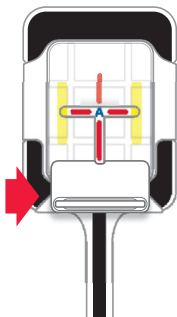
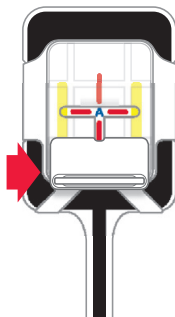
www.RinnCorp.com
www.UniGrip.com

800-323-0970
Customer Service: 847-742-1140
FAX (24 hours a day): 847-742-1122

Rinn® and Uni-Grip® are registered
trademarks of DENTSPLY International Inc.
Patent pending.
©2010 DENTSPLY Rinn. All rights reserved.
Printed in the USA

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Elgin, Illinois 60123-1819 U.S.A.